




INSTRUCTIONS FOR USE OF DISPOSABLE NEEDLE

FAST-CUT B

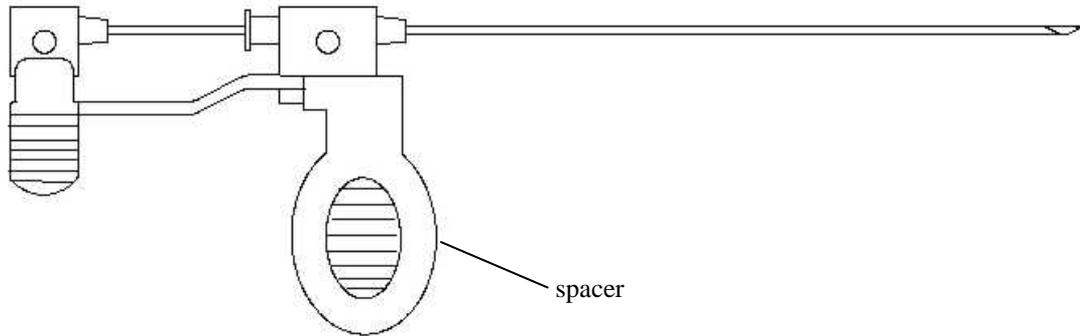

WARNING:

- Store in a dry clean place away from heat sources
- Before use ensure that the package is intact: if perforated, damaged or open discard the instrument
- Check the expiry date and that the model corresponds
- Peel open the package
- Pay attention not to compromise the sterility of the instrument before use
- In the event of a product defect or malfunction fill in the yellow claim form to be found in the product carton box and keep the defective instrument

N.B.:

The use of this instrument is reserved to qualified medical personnel aware of its application.
The instrument is sterile (sterilized by ETO GAS) single use and should be discarded after use (Circ.Min:Sanità 403/9.5/1146-L. 10.81)

SeeDOS declines any responsibility in the event of improper use or reutilisation.

The needle has been designed to be used with the automatic gun for biopsy by BARD mod. MAGNUM.

PROCEDURE

- 1) Remove the protective cover of the needle. Check the integrity of the mandrel tip and the integrity of the cannula tip (light friction between the mandrel and the cannula is normal).
- 2) Load the needle into the gun and remove the spacer by shifting it towards the tip of the needle
- 3) Use the automatic device according to the instructions given by the manufacturer of the gun.